


LOW VOLTAGE CONTROLLER MODES

FOR THE LVC-2000-007

IMPORTANT: Consult the Directions for Use for your low voltage controller for complete installation instructions

WARNING

This low voltage controller can control up to two loads independently and does NOT have multiple operational modes. The mode select switches shall be set to “0-0-0” by default. Operation with mode select switches in any other setting may warrant unwanted operation.

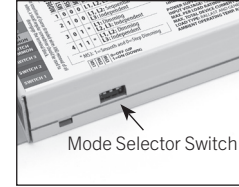

MS1 MS2 MS3 0=OFF (UP)
1=ON (DOWN)
MS1=OFF (0)
MS2=OFF (0)
MS3=OFF (0)

SWITCH 1 OPERATION

Load 1 defaults to off

- ▶ First SW1 press activates load 1
- ▶ Second SW1 press will deactivate load 1
- ▶ Subsequent SW1 presses will toggle the state of load 1
- ▶ If Load 2 is activated while trying to activate load 1, load 2 will deactivate, wait ½ second and then activate load 1

SWITCH 2 OPERATION

Load 2 defaults to off

- ▶ First SW2 press activates load 2
- ▶ Second SW2 press will deactivate load 2
- ▶ Subsequent SW2 presses will toggle the state of load 2
- ▶ If Load 1 is activated while trying to activate load 2, load 1 will deactivate, wait ½ second and then activate load 2

SWITCH 3 OPERATION

Switch 3 and Load 3 are non-functional

DIMMING

DIM1 and DIM2 are non-functional