

SENSOR PAD CLEANING INSTRUCTIONS

Clean with a non-aggressive, disinfectant cleaner. Curbell's fall management sensor pads were tested with the following chemicals. None were found to cause fading of the print or damage to the material.

NOTE: It is necessary that the manufacturer's dilution ratios and application procedures be followed.

- ▶ Advanced Sterilization Products: Cidex
- ▶ Airkem: A456-N
- ▶ Butcher's Heptagon
- ▶ Caltech: Dispatch
- ▶ Cosco: Tincture of Green Soap
- ▶ Ecolab: A-456-II Disinfectant Cleaner
- ▶ Ecolab: Oxycide Daily Disinfectant Cleaner
- ▶ Isopropyl Rubbing Alcohol 70%
- ▶ METREX: Cavi-Wipes
- ▶ METREX: Cavicide
- ▶ PDI: Super Sani-Cloth Plus Germicidal Cloth
- ▶ Spartan: SD-20
- ▶ Spartan: Hepacide Quat II
- ▶ Veridien: Viraguard Wipes
- ▶ VIROX: Accel
- ▶ Clorox: Healthcare Bleach Wipes
- ▶ Sporidicin: Sterilization Solution
- ▶ Steris: Coverage Wipes Plus
- ▶ 3M HB Quat Disinfectant Cleaner