


WIRELESS TRANSMITTER SENSOR QUICK SETUP GUIDE


NOTE: Consult the “Directions For Use” included with monitor for complete instructions, troubleshooting, and safety information




1 Turn the switch ON prior to patient use and leave it on. Once the switch has been turned on, do not turn it off.

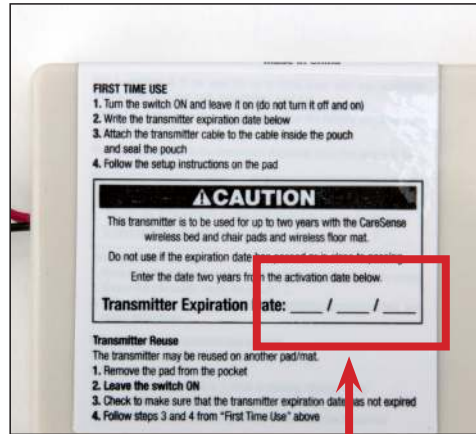


2 Write the expiration date on the transmitter

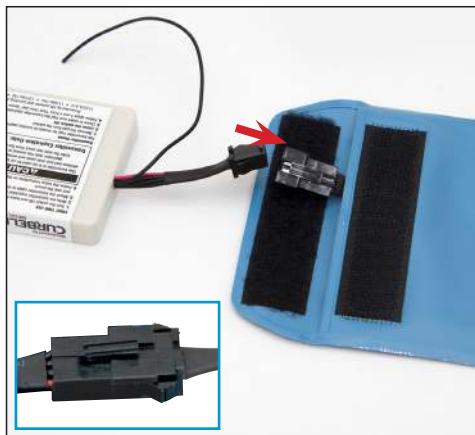

3 Attach the transmitter cable to the cable inside the pouch

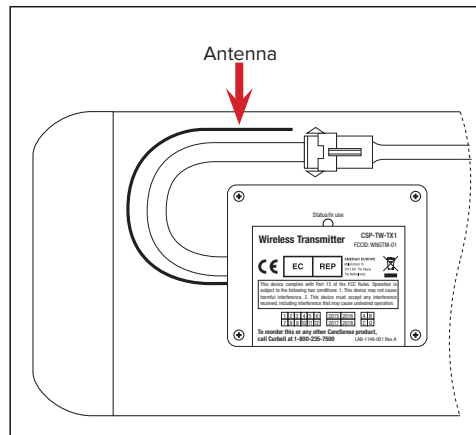

4 Place the transmitter into the pouch and seal it. Note the proper orientation of the transmitter antenna

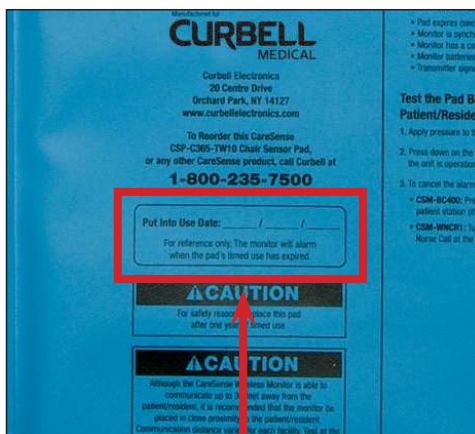

5 Write the “put into use date” on the sensor pad



6 Follow the setup instructions on the pad.