


DIRECTIONS FOR USE | Curbell Mattress for Centrella® Beds

INSTALLATION INSTRUCTIONS

1. Remove the mattress from the packaging. Inspect for shipping damage. Clean and disinfect mattress per enclosed Directions For Use.
2. Use the mattress handles to place the mattress flat on the bed with the tag end at the foot end of the bed. The bottom of the mattress has straps with clips at both ends. Unclip them both.



3. Raise the head end of the bed so you can access the bottom of the deck.
4. Insert straps through the key-hole openings on the bed deck. Click the clip ends together. There is no adjustment.

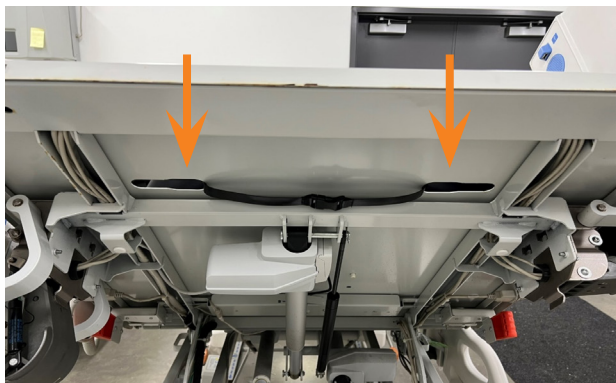

5. Lower the head end to a flat position.
6. Raise the deck to provide easier access to the bottom of the foot end bed deck.
7. Insert the straps through the key-hole openings. Click the clip ends together. There is no adjustment. The bed and mattress now are ready for use.



DISTRIBUTED BY:

CURBELL
MEDICAL

Curbell Medical Products, Inc.

20 Centre Drive • Orchard Park, NY 14127-4180

(800) 235-7500 • (716) 667-2520 • Fax: (716) 667-7775 • info@curbellmedical.com • www.curbellmedical.com