

DIRECTIONS FOR USE | Curbell Mattress Covers for Centrella® Beds

REPLACING JUST THE TOP COVER



1. Identify the head end and foot end of the mattress.



2. In the head area, pull the zipper to open it.



3. Open the inner zipper.



4. Detach both zippers.



5. Keep the bottom cover and the fire barrier.



6. Put on the new top cover. The label is at the foot.



7. Confirm the correct position for the zipper.



8. Connect the inner zipper and close it about five inches.



9. Connect the outside zipper, then fully close both zippers.

DISTRIBUTED BY:
CURBELL
MEDICAL

Curbell Medical Products, Inc.
20 Centre Drive • Orchard Park, NY 14127-4180
(800) 235-7500 • (716) 667-2520 • Fax: (716) 667-7775 • info@curbellmedical.com • www.curbellmedical.com