

REPLACING THE TOP AND BOTTOM COVERS



1. Identify the head end and foot end of the mattress.



2. In the head area, pull the zipper to open it.



3. Open the inner zipper.



4. Detach both zippers.



5. Reserve the bottom cover for removal of reusable parts.



6. Flip the mattress.



7. Remove the bed footboard anchors with a wrench.



8. Bend the plastic reinforcement in the middle area of the bottom cover to remove it from the sleeve. Set it aside for later.



9. Remove the old bottom. Install the plastic panel into the sleeve on the new cover bottom.

DIRECTIONS FOR USE | Curbell Mattress Covers for Centrella® Beds

REPLACING THE TOP AND BOTTOM COVERS (CONTINUED)



10. Flip the bottom cover face up.



11. Inspect the fire barrier for damage. Replace as needed. Identify the foot connector (glued to mattress core).



12. Install one side of the foot connector into slot in the foot end of the new bottom cover. Then install the other side.



13. Put the cover in the correct position (with head and foot at correct ends) and flip it.



14. Place top cover on top of open mattress.



15. Confirm the correct position for inner and outer zippers.



16. Close the inner zipper about five inches.



17. Connect the outside zipper and fully close both.



18. The label should be at the foot end of the bed.

REPLACING THE TOP AND BOTTOM COVERS (CONTINUED)



19. Use the mattress handles to place the mattress flat on the bed with the tag end at the foot end of the bed. The bottom of the mattress has straps with clips at both ends. Unclip them both.



20. Raise the head end of the bed so you can access the bottom of the deck. Insert straps through the key-hole openings on the bed deck. Click the clip ends together. There is no adjustment.



21. Lower the head end to a flat position. Raise the deck to provide easier access to the bottom of the foot end bed deck. Insert the straps through the key-hole openings. Click the clip ends together. There is no adjustment. The bed and mattress now are ready for use.