

# DISINFECTING INSTRUCTIONS FOR:

- **Polyurethane**
  - **Polycarbonate**
  - **Lectrolite®**
- 

Wipe down with one of the non-aggressive, disinfectant cleaners listed below. Curbell's mattress covers were tested with these cleaners, and none were found to cause fading or damage to the material. If your disinfectant is not listed below, please contact Curbell.

- Advanced Sterilization Products: Cidex®
- Airkem: A456-N
- Caltech®: Dispatch®
- Clorox®: Healthcare Bleach Wipes\*
- Cosco: Tincture of Green Soap
- Diversey: Oxivir® Tb
- Ecolab®: A-456-II Disinfectant Cleaner
- Isopropyl Rubbing Alcohol 70%
- METREX: Cavi-Wipes™
- METREX: Cavicide™
- Misco Products Corp. (MPC): Hi-Con PF3
- PDI®: Super Sani-Cloth® Plus
- Spartan®: SD-20
- Spartan: Hepacide Quat II
- Sporidicin®: Sterilization Solution
- Steris: Coverage Wipes Plus
- Stryker®: Sidekick Disinfecting Wipes
- Veridien: Viraguard® Wipes
- VIROX®: Accel
- VIROX: 256
- Wayne®: Concept 256N Disinfectant

## NOTE:

- **It is necessary that the disinfectant manufacturer's dilution ratios and application procedures be followed.**
- **After the disinfectant manufacturer's dwell time has been achieved, the surface must be immediately and thoroughly rinsed with warm water.**
- **Do not allow the disinfectant to pool on the surface longer than the disinfectant manufacturer's dwell time.**
- **Do not allow the disinfectant to dry on the surface.**

\* It is not recommended to use bleach on polyurethane